



Convenience Kit for Collecting and Transporting Anterior Nares Specimens

Re-order No. K165

For in vitro Diagnostic Use - Rx Only

INTENDED USE

Quest Diagnostics Convenience Kit for Collecting and Transporting Anterior Nares Specimens is intended for use by healthcare professionals (or by patients self-collecting in the physical presence of an observing healthcare professional) to collect and transport a nasal (anterior nares) specimen to Quest Diagnostics for testing.

Not for at-home self-collection.

THIS COLLECTION KIT IS NOT APPROPRIATE FOR EVERY TEST. PLEASE SEE THE SPECIMEN COLLECTION GUIDE FOR THE RELEVANT TEST AT <https://testdirectory.questdiagnostics.com/test/home>.

MATERIAL PROVIDED

1. Phosphate Buffered Saline Transport Tube (PBS Transport Tube). Four tubes. Each tube contains 2 mL of phosphate buffered saline (PBS). Ready for use and no further preparation is necessary. The product should be stored in its original container at 4 °C to 25 °C until used. Do not overheat. Do not incubate, or freeze prior to use. Improper storage may result in a loss of efficacy. This product is sterilized by E-Beam.
2. Nasal (anterior nares) Swabs (Swab). Four swabs. Swabs may be either a nylon or polyester flocced or spun swab. The Swab will NOT have a wooden shaft, and will NOT contain calcium alginate or bacterial culturette-type liquid or gel transports. The product should be stored in its original container at 4 °C to 25 °C until used.

WARNINGS AND PRECAUTIONS

1. For in vitro diagnostic use only.
2. Consider all human specimens, blood derivatives, reagents and materials used for processing as capable of transmitting infectious diseases and handle them in a manner which prevents infection of personnel. No known test method can offer complete assurance that infectious agents are absent.
3. Observe approved biohazard precautions and aseptic techniques. To be used only by adequately trained and qualified personnel. Wear suitable protective clothing, gloves, and eye/face protection when handling the contents of this kit. Wash hands thoroughly after handling.
4. Pathogenic microorganisms, including hepatitis viruses (eg, HBV, HCV), human immunodeficiency virus (eg, HIV-1 and HIV-2), Human T-cell lymphotropic virus (HTLV, types I and II), STS, etc, may be present in clinical specimens. "Standard Precautions" and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids. Sterilize all biohazard waste including specimens, containers, and media after use. Dispose of containers and unused contents in accordance with Federal, State and Local regulatory requirements.
5. Directions must be read and followed carefully.
6. Single Transport Tubes are not intended for use with multiple patients.
7. Do not re-pack.
8. Do not ingest the PBS. Avoid contact with skin, mucosal membranes and eyes with the PBS. Do not use the PBS to moisten nasal passages. Do not use the PBS for pre-moistening or pre-wetting the applicator swab prior to collecting the sample or for rinsing or irrigating the sampling sites.

PROCEDURE

1. Write the patient's name, date of birth, date of collection and other information on the PBS Transport Tube.
2. Collect the specimen according to standard specimen collection practices.
3. Place the Swab in the PBS Transport Tube. If the Swab is too long to fit into the PBS Transport Tube, then break off the Swab at the perforation.
4. Securely attach the cap to the PBS Transport Tube. Note: the PBS Transport Tube has a "screw cap" (not a push cap). Please ensure the screw cap is gently turned in clockwise rotation until seated (firm stop).
5. Package the PBS Transport Tube in a secondary container, for example a Ziploc[®] bag, along with a completed test requisition. See the specimen collection guide for the relevant test at www.questdiagnostics.com for the appropriate storage temperature and speed for shipping the specimen.

Do not use PBS Transport Tube or Swab if:

- There is evidence of damage or contamination to the product;
- The PBS Transport Tube has evidence of leakage, the cap does not fit properly, or the tube or cap is cracked or misshaped;
- The color of the PBS is no longer clear and transparent;
- The expiration date has passed;
- There are other signs of deterioration or the integrity of the Swab's packaging has been compromised.

ASSISTANCE

To place an order, please contact a Quest Diagnostics Representative at 1.866.697.8378.

For kit problems, please call 1.866.697.0040 or contact us at QuestConvenienceKit@questdiagnostics.com.

LabOne, Inc. (a subsidiary of Quest Diagnostics)
10101 Renner Blvd
Lenexa KS

QuestDiagnostics.com Quest, Quest Diagnostics, any associated logos, and all associated Quest Diagnostics registered or unregistered trademarks are the property of Quest Diagnostics. All third-party marks—® and ™—are the property of their respective owners. ©2020 Quest Diagnostics Incorporated. All rights reserved. rev 06/17/2020